



INSTRUCTIONS FOR USE

Product Name

OneCath® Lubricated Intermittent Urinary Catheter

Indication

OneCath® Lubricated Intermittent Urinary Catheter is indicated for use in male, female, and pediatric patients [adolescents and transitional adolescents] with chronic urine retention and patients with post void residual volume [PVR] due to neurogenic and non-neurogenic voiding dysfunction. The catheter is inserted into the urethra to reach the bladder allowing urine to drain.

How to Use

Wash your hands with soap and water and dry thoroughly before preparing for catheterization.

Gently clean your urethral opening with a non-alcohol based antiseptic wipe. Open the peelable package, pick up the catheter, connect adapter with extension tube as needed. Pull back the sleeve to expose the catheter tip. Insert the catheter through the urethra, do not stop inserting until urine comes out. **Do not** allow the sleeve to enter the urethra. After drainage, pull the catheter out of the urethra.

Disposal

Follow local governing ordinances and recycling instructions regarding disposal or recycling of the device and components.

Warnings and Cautions



Federal law restricts this device to be sold by or on the order of a physician. OneCath® urinary catheter is for single use only, and must then be discarded. CompactCath Inc. accepts no liability for injury or loss resulting from use of this product in any manner if the catheter is not used entirely in accordance with the company's recommendations.


Self-catheterization is a safe and common procedure. However, it is important that prior to using OneCath® urinary catheter you would consult a medical professional for guidance and that you follow these instructions carefully. Self-catheterizing using OneCath® urinary catheter should only be done on the order of a physician/nurse.

If you experience the catheter does not drain your bladder the way you are used to, please contact a physician/nurse. In case of Urinary Tract Infection with symptoms such as fever, discomfort when emptying the bladder, a frequent urge to urinate or blood in the urine, contact your physician/nurse.

If the package of the catheter is not sealed, or if the catheter is otherwise damaged, do not use the catheter. Store product at room temperature.

Definition of Symbol

-  Consult instructions for use
-  Keep away from sunlight
-  Use-by date
-  Keep dry
-  Do not re-use
-  Do not use if package is damaged
-  Batch code
-  Prescription only
-  Sterilized using ethylene oxide

 Manufacturer: CompactCath Inc.
887 Federation Way, Palo Alto, California 94303 U.S.A.
Customer Service: 1-888-933-2284 [WEE- CATH]